

A stylized boomerang graphic composed of multiple overlapping curved lines in shades of light blue and green, arching over the top of the page.

Boomerang GEL PAD™


Returning Compliance to Every Patient



AG
INDUSTRIES
delivering pure™

Medium/Large
AGGELPAD-ML

cpap-online.de

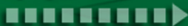



TO USE

1. Before each use wash the Boomerang Gel Pad™ in warm water using mild soap. Rinse thoroughly with potable water before applying to skin.
2. Place the Boomerang Gel Pad™ on bridge of the nose as shown.
3. Fit your mask as usual over the pad.
4. Replacement recommended every 30 days.

RECOMMENDED CLEANING

Wash in cold or warm water using mild soap. Rinse with potable water and allow to air-dry. Keep out of direct sunlight. May be cleaned with alcohol-based hand sanitizer.

CAUTION

Do not clean with shampoo.
Do not place in microwave.
Do not place in dishwasher.
Do not ingest or place in mouth.
Keep away from open flames.
Keep out of direct sunlight.

MATERIAL

Polymer gel composed from a proprietary USP grade mineral oil including moisturizing essential oils. Non-toxic, fully washable, hypoallergenic. Latex and silicone-free.

STORAGE

Store only in original package.
Do not store or place on any other surface.



LOT	318RF
REF	AGGELPAD-ML

MANUFACTURED FOR
AG Industries, Inc
3637 Scarlet Oak Blvd.
St. Louis, MO 63122
MADE IN TAIWAN

